

Easy step by *Step*  
*by step* guide


**MATERIALS**  
**Handling** *Working with ease...*  
PTY LTD

Our website: [www.materialshandling.com.au](http://www.materialshandling.com.au)  
Phone us: 1300 25 84 07 Fax us: 1800 68 68 96  
Email us: [sales@materialshandling.com.au](mailto:sales@materialshandling.com.au)

**Australia Wide Sales and Service**

# Step 1:

Unplug charger  
from  
EVO-MOVER

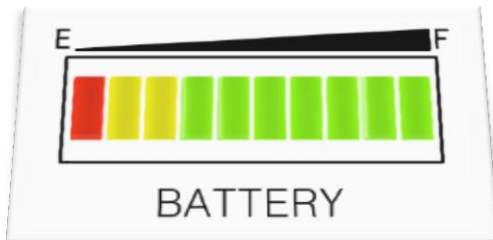

# Step 2:

Understand  
your basic  
controls.




# Step 2: Continued

**- DRIVE LEVER-  
DISENGAGES AND  
ENGAGES MOTOR**

While lever is UP, the unit can be pushed manually.  
When DOWN, the motor controls forward /reverse



**"UP"  
DISENGAGES  
DRIVE**

**"DOWN"  
ENGAGES**

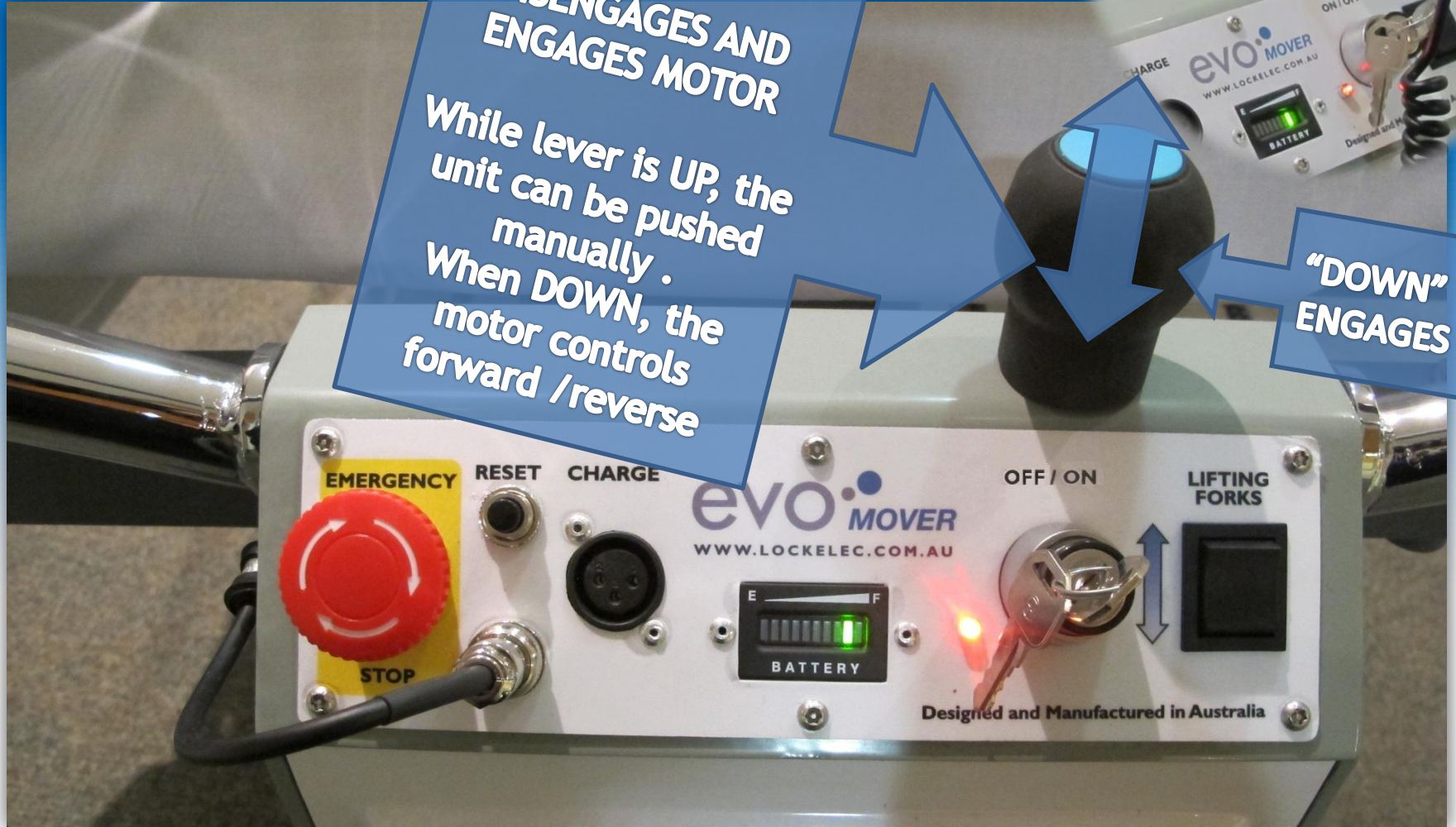

# Step 2: Continued


DRIVE  
LEVER

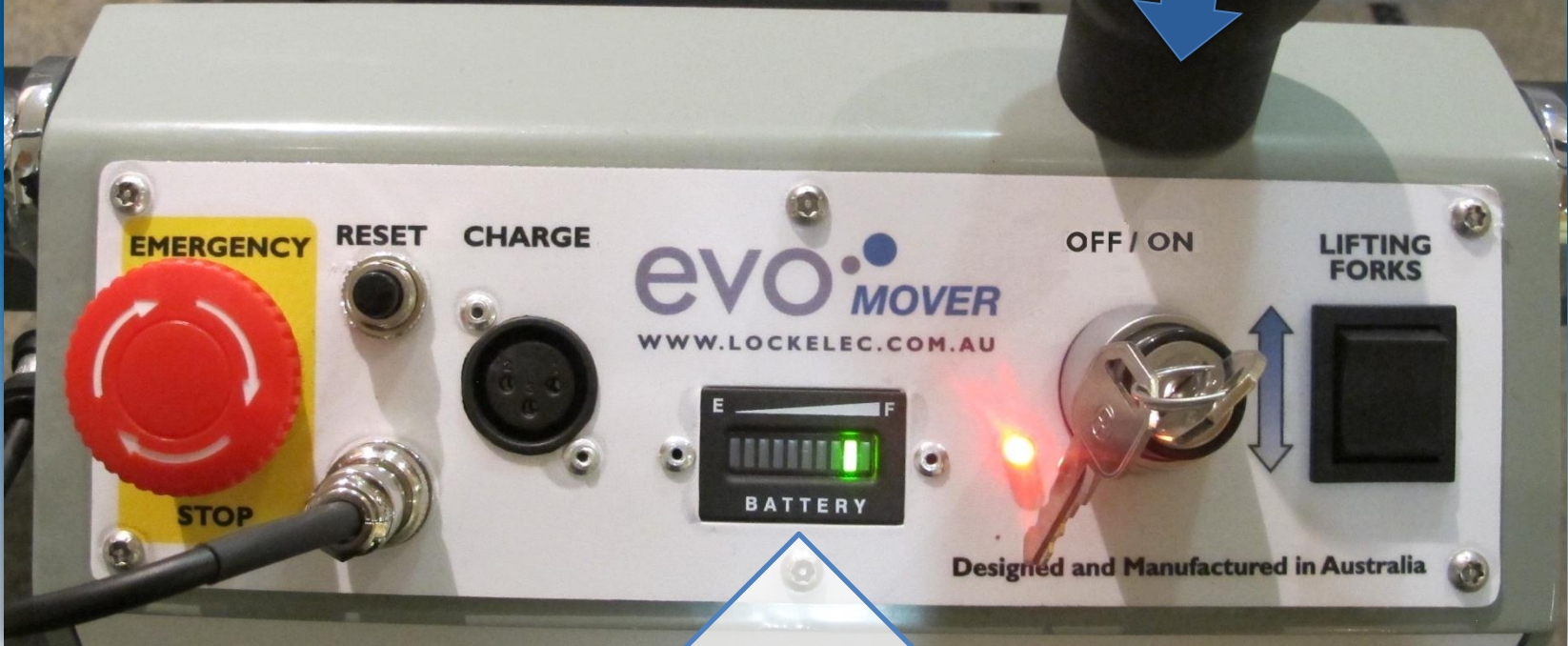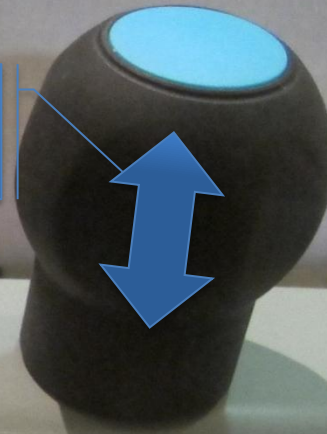

E

F

**GREEN** INDICATES THE EVO MOVER IS CHARGED  
**AMBER** REQUIRES CHARGING IMMEDIATELY  
**RED** IS DANGEROUSLY LOW, STOP AND CHARGE

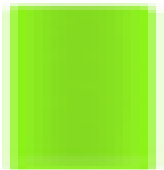

# Step 3:



## Attaching:



Ideally the bed must be brought out into the hallway before attaching if the bed is located in a confined room.



# Step 3: continued

Manually push mover in and under the bed and align the grasping forks with bed frame.

SLIDE V-BLOCKS FORWARD AND BACKWARDS TO CREATE A NICE CLOSE PROFILE FIT BETWEEN THE EVO MOVER AND THE BED.  
(This will assist in tight lifts)

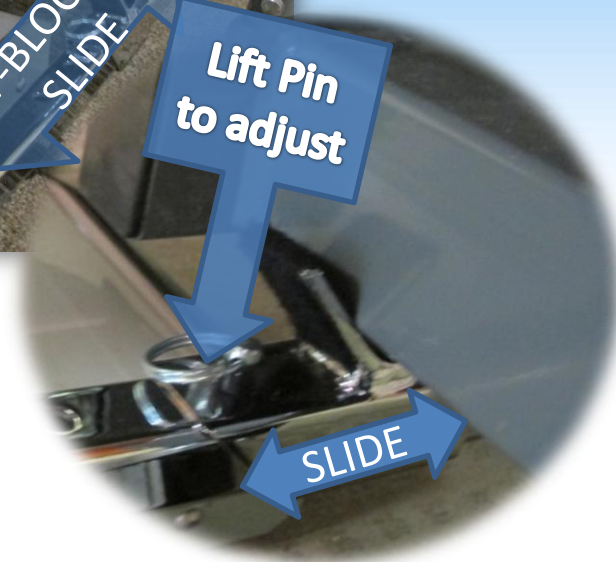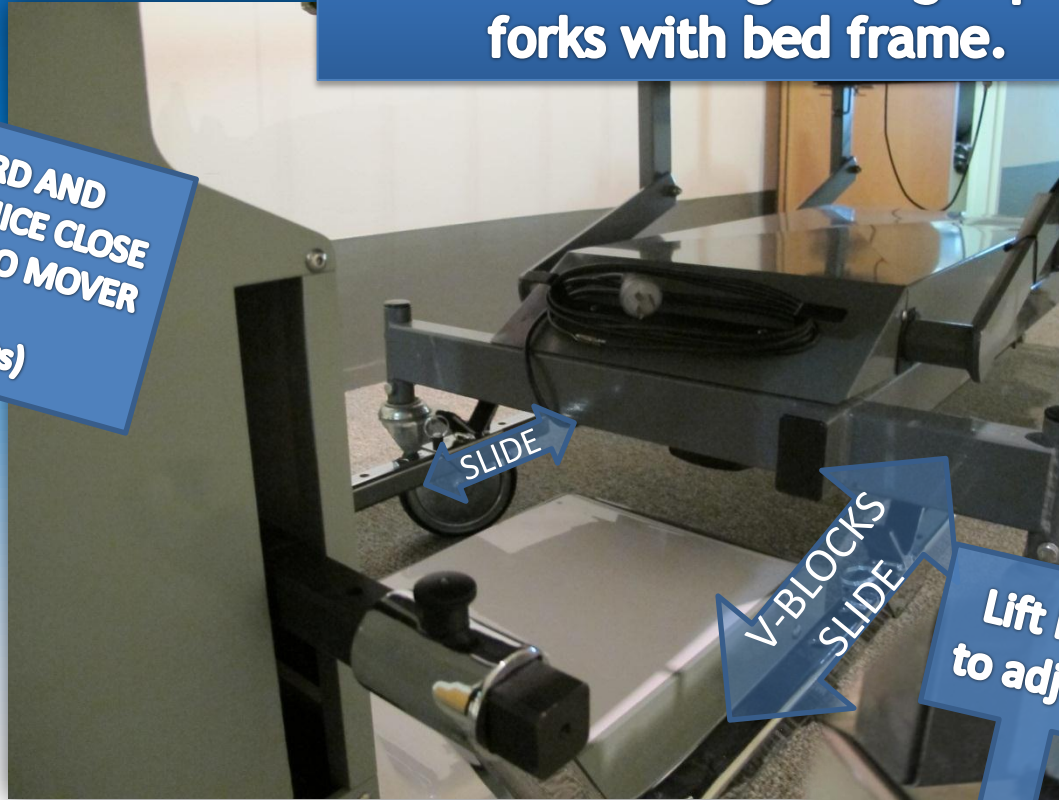

# Step 3: continued

Push button UP to raise forks





The forks can lift the bed off the ground.

Stop at **1CM** in height



# Step 3: continued

Only if required

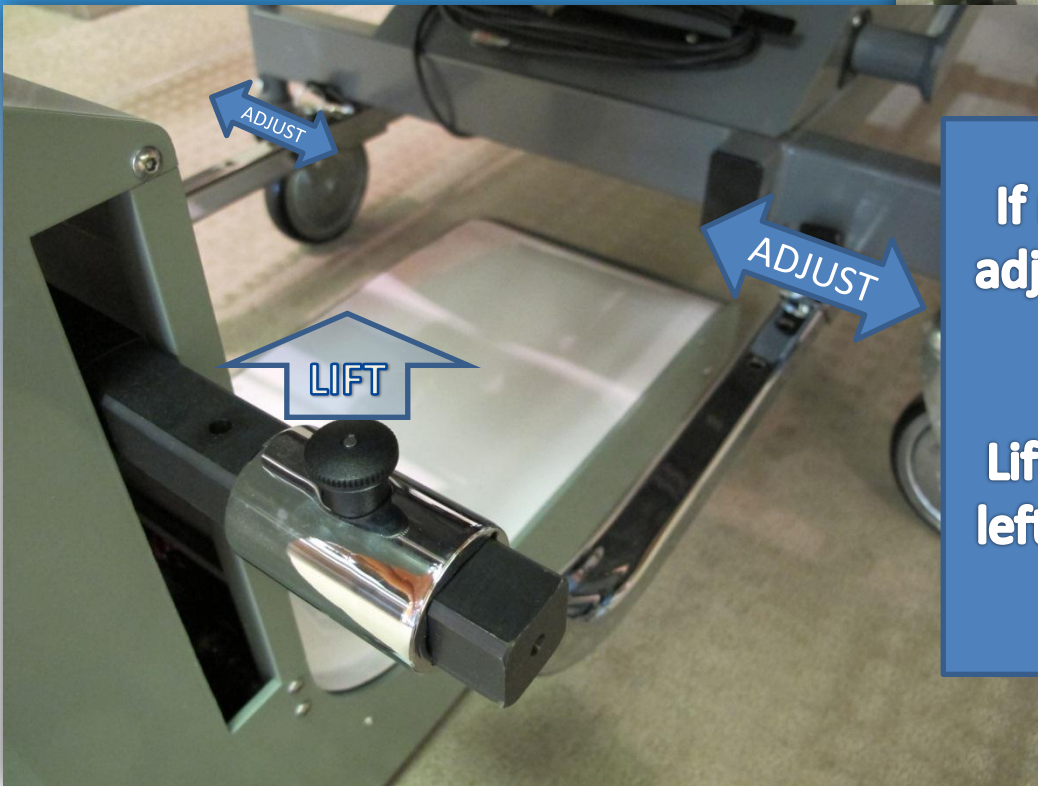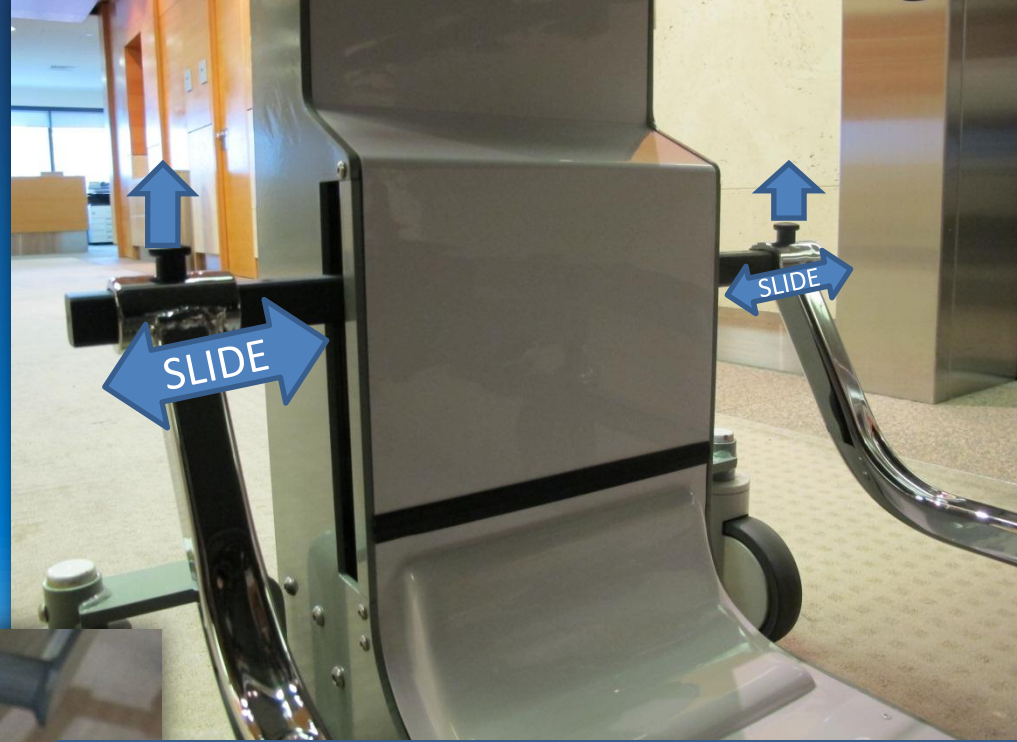

If required the Evo-Movers arms can be adjusted in width to achieve an improved grasping point to the bed.

Lift the Index plunger and slide the arms left or right until the plunger retracts to a locking point.

# Step 4:

Engage drive  
lever into  
Drive

**PUSH DOWN**



# Step 5:

TAKE HANDSET  
AWAY FROM THE  
UNIT



Walk to the rear of  
the bed with the  
HANDSET





FORWARD

# Step 6:

If you are turning right position yourself as far right in the corridor when approaching the corner.


*The opposite applies when turning left.*

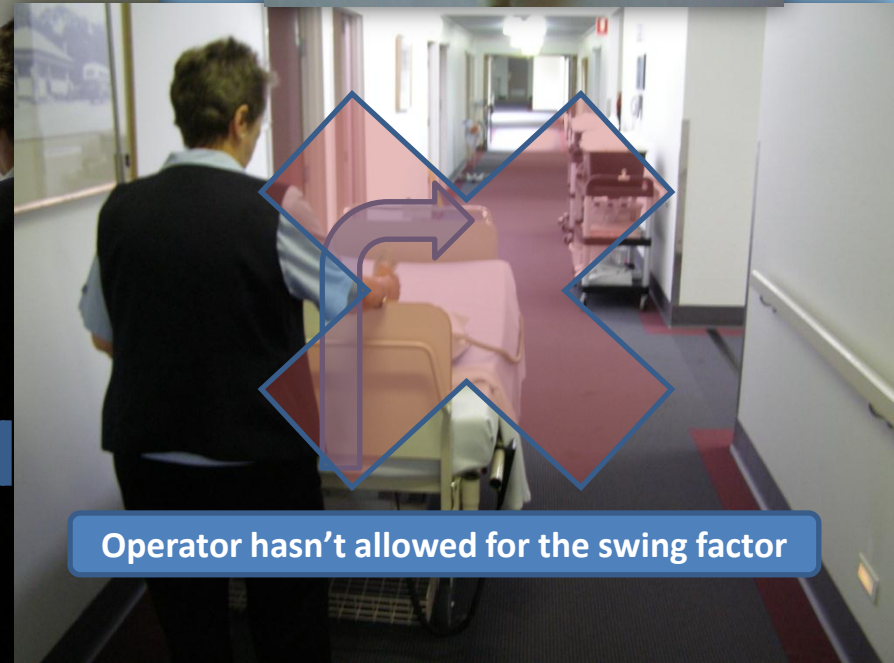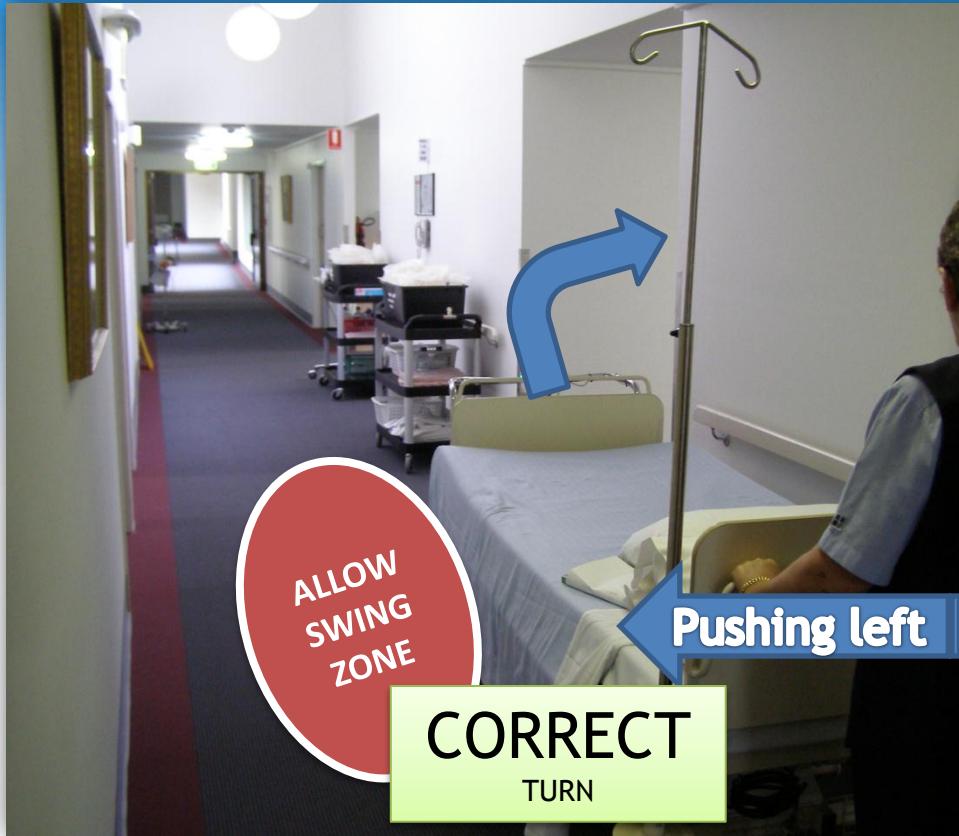

# Step 7:

## DISENGAGING

Push button  
DOWN to bring  
the forks off

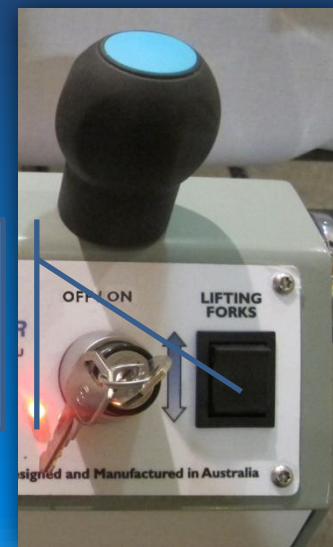

# Step 8: continued

Ensure the grasping forks have travelled downwards to clear the beds frame



Handset has been brought back




LIFT drive lever UP and simply pull the unit away

# Step 9:

**DON'T FORGET**

Always put the  
Evo Mover back on  
charge when not in use

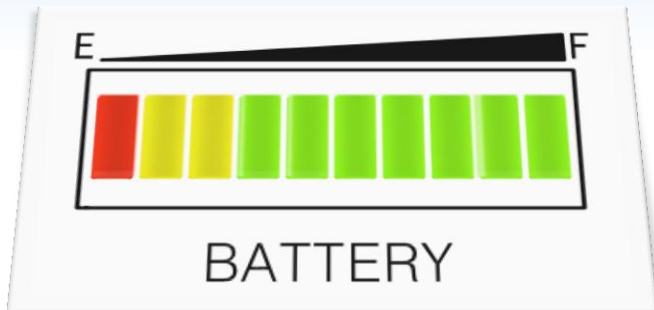

**MATERIALS**  
**Handling** *Working with ease...*  
PTY LTD

evo<sup>•</sup>MOVER